

Quick guide to install the **OKTAGON**[®] libraries

3shape

1. Download and open the file „3SHAPE_libraries_V1.2.zip”
2. Unzip the file „MEISINGER IMPLANTS 3SHAPE libraries v1.2.dme“, with the **OKTAGON**[®] components of library
3. Open the Dental System Control Panel and go to „Tools“ -> „Import/Export” and choose „Import materials”
4. Now select the unzipped „*.dme“ file for importing
5. Done, the 3shape library is now available for all **OKTAGON**[®] implants

dental wings

1. Download and open the file „Dental_wings_libraries_V1.2.zip”
2. Unzip the folder „Meisinger Implants DWOS V1.2“ with the **OKTAGON**[®] components of library
3. Start the program Dental Wings and choose the module „Implant Kits“.
4. On the chosen module „Implant Kit Editor“ do a right mouse click.
5. Choose with a left mouse click the menu item „Import Implant Manufacturer Kits“
6. Choose the file „Library V1.2.zip“
7. Done, the dental wings library is now available for all **OKTAGON**[®] Implants

exocad

1. Download and open the file „Exocad libraries V1.2.zip”
2. Open the folder „Meisinger Implants EXOCAD V1.2“ and unzip the file „Meisinger Implants Library V1.2.zip „ in the folder DentalCADApp\library\implant
3. Done, the exocad library is now available for all **OKTAGON**[®] Implants